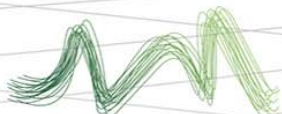
Dental hard- and soft tissue regeneration

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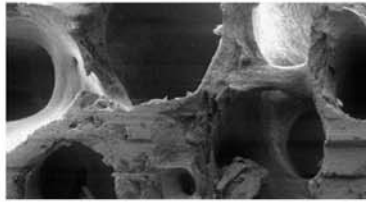
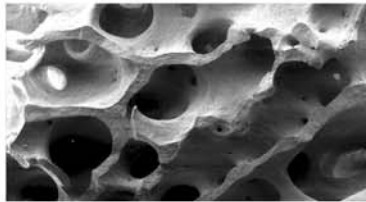
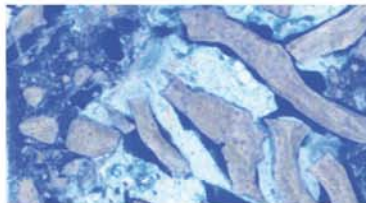
Dentegris Service & Logistic Center

Grafshafter Straße 136
service@dentegris.com

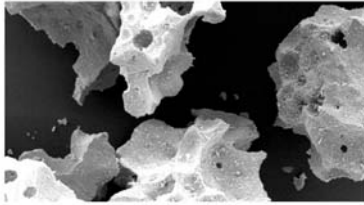
DE-47199 Duisburg
www.dentegris.com


CompactBone S.

Innovative, biphasic resorbable calcium phosphate

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
SEM: CompactBone S. Granules

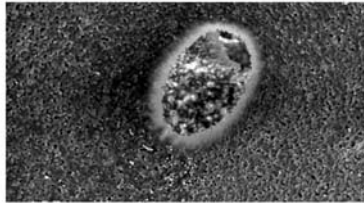SEM: Interconnecting window

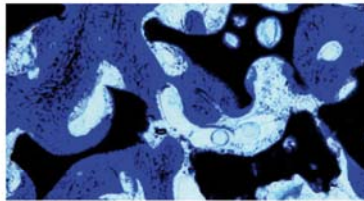Histology of CompactBone S. 6 months after sinus lift: optimal integration and osseous integration

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

BoneProtect® Membrane

Collagen membrane - highly effective,
resorbable natural soft tissue

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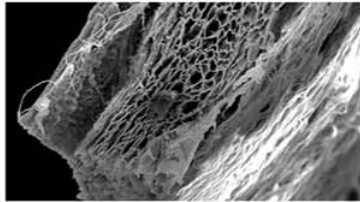
SEM: BoneProtect® Membrane

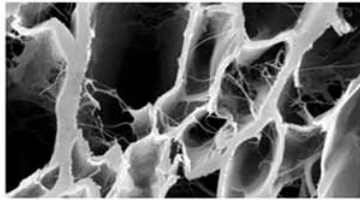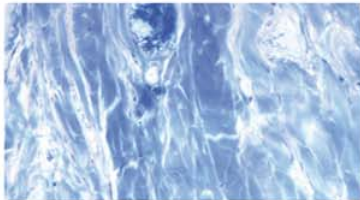SEM: 3D structure of BoneProtect® Membrane

Histology of BoneProtect® Membrane 4 weeks after implantation: perfect integration without inflammatory reaction; the structure of the membrane is clearly recognizable

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Dentegris Service & Logistic Center



Grafshafter Straße 136
service@dentegris.com

| DE-47199 Duisburg
| www.dentegris.com


BoneProtect® Guide

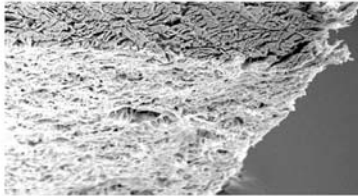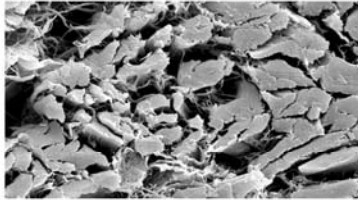Naturally cross linked collagen membrane

SEM: BoneProtect® Guide

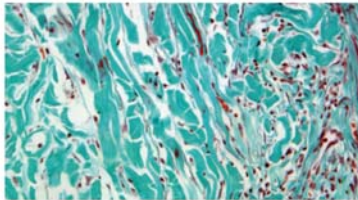Histology 6 weeks after implantation of BoneProtect® Guide: blood vessels have penetrated the porous structure, collagen fibres are visible and the resorption process is ongoing without any inflammatory tissue response.

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

BoneProtect® Fleece

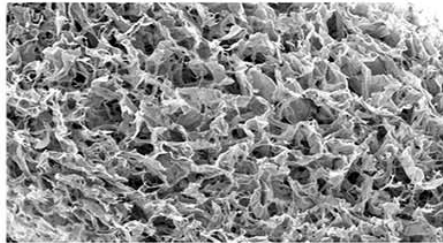Collagen fleece - highly effective,
resorbable natural soft tissue

SEM: BoneProtect® Fleece

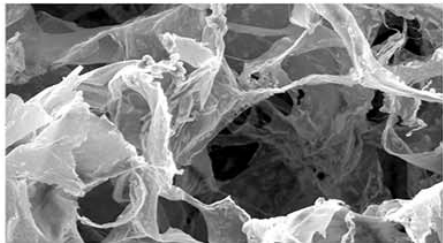SEM: 3D structure of BoneProtect® Fleece

Clinical application of BoneProtect® Fleece

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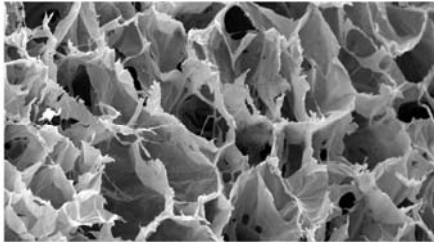
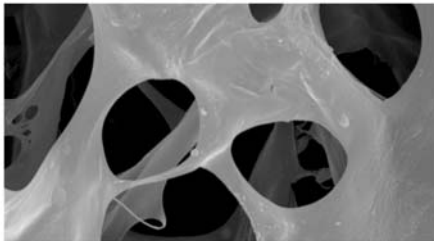
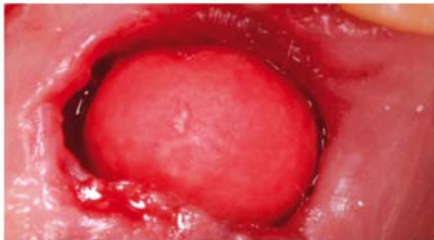
BoneProtect® Cone


Item no.	Size	Content
CONE-12	16mm height with diameter on top ~11mm and bottom ~7mm	12 pieces (single sterile units)

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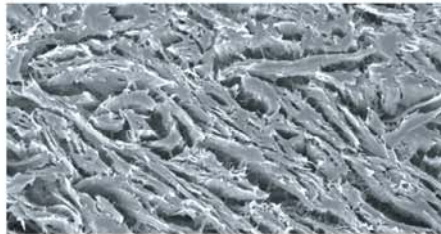
REM: MucoMatrixX[®]

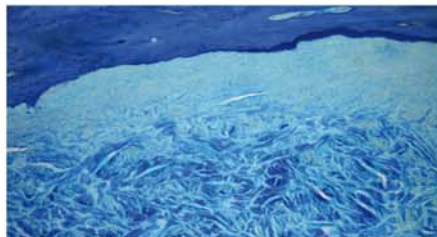MucoMatrixX[®] good handling after rehydration with blood.

Histology of MucoMatrixX[®] 6 months after implantation. Optimal integration without inflammatory reaction with the MucoMatrixX[®].

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
Root coverage: MucoMatrixX® case by Dr. Roland Török, Nürnberg

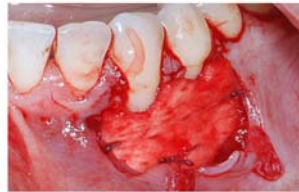Preclinical situation, root recession visible.

Clinical situation after MucoMatrixX® placement.

1 month post op.

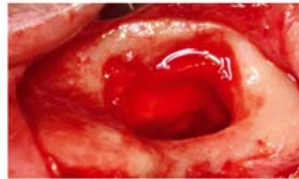
2 years post op


Sinuslift with lateral approach: Case by Dr. Stefan Peev, Sofia

Insertion of BoneProtect® Fleece.

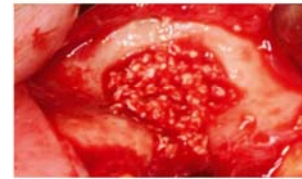BoneProtect® Fleece placed on Sinus membrane.

The implant is placed and the grafting cavity is filled up with CompactBone B..

BoneProtect® Membrane is placed to close the lateral window.

Socket preservation: Case by Dr. Fernando Rojas-Vizcaya, Castellón & Chapel Hill

Clinical situation after tooth removal.

Socket filled with CompactBone B. and buccal wall protected with BoneProtect® Guide.

BoneProtect® Guide turned down over the socket.

Final suturing of the socket.

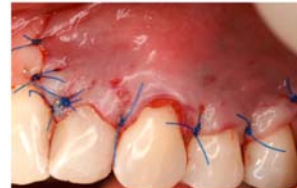
Root recession: MucoMatrixX[®] case by PD Dr. Stefan Hägewald, Berlin

Clinical situation of the root recession before MucoMatrixX[®] placement.

MucoMatrixX[®] placement over the tooth root.

Gingival tissue was coronally repositioned, covering the MucoMatrixX[®] and the roots of teeth, and sutured in place.

6 months post-op view: the previously recessed roots of teeth are covered with attached pink, keratinised gingival tissue.

Root coverage: MucoMatrixX[®] case by PD Dr. Adrian Kasaj, Mainz

Clinical situation of the root recession before MucoMatrixX[®] placement.

MucoMatrixX[®] placement over the teeth root area.

Gingival tissue was coronally repositioned, covering the MucoMatrixX[®] and the roots of teeth, and sutured in place.

6 months post-op view: the previously recessed roots of teeth are covered with attached pink, keratinised gingival tissue.

Horizontal & vertical augmentation: CompactBone B; BoneProtect[®] Membrane case by Dr. Marius Steigmann, Neckargemünd

Clinical application after implant insertion.

Horizontal and vertical augmentation with CompactBone B., particle size 1-2 mm.

Coverage of the augmentation with BoneProtect[®] Membrane, size 20 x 30 mm.

Tension-free wound closure.